All copy and logo's print 100% black unless stated

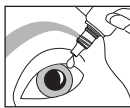
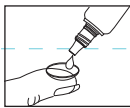
blink

contacts®

TOP FRONT

soothing eye drops

10ml

(EN) Refreshment for contact lens wearers from dry, tired eyes

Specifically formulated for contact lens wearers this gentle and highly effective eye drop soothes and refreshes dry, tired eyes. Suitable for all contact lenses without the need to remove your lenses first.

Instructions for use

- Place 1 or 2 drops in affected eye as required
- Blink several times to wash the eye
- Use as often as required
- No need to remove your lenses
- For extra comfort place 1 or 2 drops on each lens before inserting

For use with all types of contact lenses.

Precautions

- Do not use if the tamper-evident seal is broken or missing
- Do not swallow the solution
- Do not use if allergic to any of the ingredients
- To avoid contamination do not touch bottle tip on any surface and avoid contact with eyes
- Close the bottle tightly immediately after use
- Do not use after the expiry date
- Keep out of the reach of children
- Store between 5°C and 25°C
- Do not rinse your lenses in tap water
- Discard any remaining solution 45 days after first opening the bottle

If you have continued problems please discuss with your eye care professional.

Contains:

Sodium hyaluronate - 0.15%
 OcuPure® preservative - 0.005%
 Calcium chloride (dihydrate)
 Potassium chloride
 Sodium chloride
 Borate buffer
 Magnesium chloride
 Purified water

Symbols

Consult instructions for use	Manufacturer	5°C - 25°C Temperature limitation
STERILE A Sterile-Aseptic technique	LOT Batch code	Discard any remaining solution 45 days after first opening the bottle
Do not use if package is damaged	Use by	

CE 0123 STERILE A

AMO Ireland
 Block B, Liffey Valley Office Campus
 Quarryvale, Co. Dublin, Ireland
 blink, blink contacts and OcuPure are trademarks of Johnson & Johnson Surgical Vision, Inc.
 © Johnson & Johnson Surgical Vision, Inc., 2018

CE mark size/proportions complies with European Medical Devices Directive 93/42/EEC Annex XII

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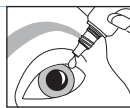
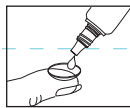
AM70477AK13D
 Revision Date: 04/2018
 XXXXXX XX

10ml INSERT (AUTO)
 207mm x 117mm
 AM01194

ENTIRE INSERT TO PRINT OUT OF PROCESS BLACK

<i>DC • Labeling & Packaging</i>																			
Artwork Specifications																			
Platform: MAC - Adobe Illustrator CS4	- ARTWORK IS ACTUAL SIZE. - DROP KEYLINES AND CALLOUTS BEFORE PROCESSING. - ALL COPY MUST BE CLEAR & LEGIBLE. - SMEARING/BROKEN TYPE IS NOT ACCEPTABLE.																		
Last Update: 17/04/2018 - 11.30 v1																			
Note: This colour copy is to show size, copy placement and colour breaks. Actual colours will be matched on press to approved colour standards and/or PMS colour swatches.	<table border="1" style="width: 100%; border-collapse: collapse;"> <tr> <td style="width: 20px; text-align: center; font-size: 8px;">INK(S)</td> <td style="width: 20px; height: 20px;"></td> <td style="width: 20px; height: 20px;"></td> <td style="width: 20px; height: 20px;"></td> <td style="width: 20px; height: 20px;"></td> <td style="width: 20px; height: 20px; background-color: black;"></td> <td style="width: 20px; height: 20px;"></td> <td style="width: 20px; height: 20px;"></td> <td style="width: 20px; height: 20px;"></td> </tr> <tr> <td></td> <td></td> <td></td> <td></td> <td style="text-align: center; font-size: 8px;">BLACK</td> <td></td> <td></td> <td></td> <td></td> </tr> </table>	INK(S)													BLACK				
INK(S)																			
				BLACK															