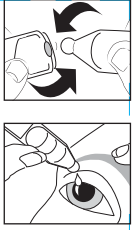
FRONT COPY

TEXT DIRECTION

blink[®]
intensive
soothing eye drops
20 x 0.4ml

(EN) Long lasting relief from the feeling of dry, tired eyes
An advanced lubricating eye drop specifically formulated to provide soothing comfort for eyes that are dry, irritated, or uncomfortable.

Indications
Irritated, uncomfortable eyes. May be used with contact lenses.

Contraindications
No known contraindications.

Instructions for use

- Place 1 or 2 drops in affected eye as required
- Blink several times to wash the eye
- Use as often as required
- If you wear contact lenses there is no need to remove them

Precautions

- Always use fresh solution
- Do not use if unit dose container is damaged
- Do not swallow the solution
- Do not use if allergic to any of the ingredients
- To avoid contamination do not touch container tip on any surface and avoid contact with eyes
- Do not use after the expiry date
- Keep out of the reach of children
- Store between 5°C and 25°C
- Discard the container and any remaining solution immediately after use
- Potential risks, side effects or complications generally associated with eye drops are:
Allergic reaction, inflammatory reaction or infections

If you have continued problems please discuss with your eye care professional.

Side Effects
No known side effects.

Contains:
Polyethylene glycol 400 (PEG 400) – 0.25%
Sodium hyaluronate – 0.20%
Boric acid
Sodium borate (decahydrate)
Sodium chloride
Potassium chloride
Calcium chloride (dihydrate)
Magnesium chloride
Purified water

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CE 0123 **STERILE A**

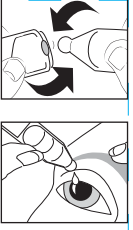
AMO Ireland
Block B, Liffey Valley Office Campus
Quarryvale, Co. Dublin, Ireland
blink is a trademark of Johnson & Johnson Surgical Vision, Inc.
© Johnson & Johnson Surgical Vision, Inc. 2018

CE MARK SIZE/PROPORTIONS COMPLIES WITH
EUROPEAN MEDICAL DEVICES DIRECTIVE
93/42/EEC Annex XII.

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INSERT 240mm x 140mm
(HOLOPACK)
PENCIL FOLD

AM01699

ENTIRE INSERT TO PRINT OUT OF PROCESS BLACK

DC • Labeling & Packaging																									
Artwork Specifications																									
Platform: MAC - Adobe Illustrator CS4	- ARTWORK IS ACTUAL SIZE. - DROP KEYLINES AND CALLOUTS BEFORE PROCESSING. - ALL COPY MUST BE CLEAR & LEGIBLE. - SMEARING/BROKEN TYPE IS NOT ACCEPTABLE.																								
Last Update: 03/05/2018 - 10.00 v1																									
Note: This colour copy is to show size, copy placement and colour breaks. Actual colours will be matched on press to approved colour standards and/or PMS colour swatches.	<table border="1" style="display: inline-table; border-collapse: collapse;"> <tr> <td style="width: 20px; height: 20px;"> </td> <td style="width: 20px; height: 20px;"> </td> <td style="width: 20px; height: 20px;"> </td> <td style="width: 20px; height: 20px;"> </td> <td style="width: 20px; height: 20px;"> </td> <td style="width: 20px; height: 20px;"> </td> <td style="width: 20px; height: 20px;"> </td> <td style="width: 20px; height: 20px;"> </td> </tr> <tr> <td style="text-align: center;">INK(S)</td> <td></td> <td></td> <td style="background-color: black;"></td> <td></td> <td></td> <td></td> <td></td> </tr> <tr> <td></td> <td></td> <td></td> <td style="text-align: center;">BLACK</td> <td></td> <td></td> <td></td> <td></td> </tr> </table>									INK(S)											BLACK				
INK(S)																									
			BLACK																						