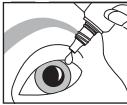
All copy and logo's print 100% black unless stated

**blink®**  
**intensive**  
soothing eye drops  
10ml

**TOP FRONT**

**(EN)** Long lasting relief from the feeling of dry, tired eyes

An advanced lubricating eye drop specifically formulated to provide soothing comfort for eyes that are dry, irritated, or uncomfortable.

**Indications**  
Irritated, uncomfortable eyes. May be used with contact lenses.

**Contraindications**  
No known contraindications.

**Instructions for use**

- Place 1 or 2 drops in affected eye as required
- Blink several times to wash the eye
- Use as often as required
- If you wear contact lenses there is no need to remove them

**Precautions**

- Do not use if the tamper-evident seal is broken or missing
- Do not swallow the solution
- Do not use if allergic to any of the ingredients
- To avoid contamination do not touch bottle tip on any surface and avoid contact with eyes
- Do not use after the expiry date
- Keep out of the reach of children
- Store between 5°C to 25°C
- Discard any remaining solution 45 days after first opening the bottle
- Potential risks, side effects or complications generally associated with eye drops are: allergic reaction, inflammatory reaction or infections

If you have continued problems please discuss with your eye care professional.

**Side Effects**  
No known side effects.

**Contains:**  
Polyethylene glycol 400 (PEG 400) – 0,25%  
OcuPure® preservative (sodium chlorite)  
Sodium hyaluronate – 0,20%  
Boric acid  
Sodium borate (decahydrate)  
Sodium chloride  
Potassium chloride  
Calcium chloride (dihydrate)  
Magnesium chloride  
Purified water

**PAGE 1**

**Symbols**

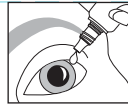
Consult instructions for use	Manufacturer	25°C Temperature limitation	
<b>STERILE A</b> Sterile, Aseptic technique	<b>LOT</b> Batch code	5°C	
Do not use if package is damaged	Use by	45D	Discard any remaining solution 45 days after first opening the bottle

**CE 0123** **STERILE A**

AMO Ireland, Block B, Liffey Valley Office Campus  
Quarryvale, Co Dublin, Ireland  
blink and OcuPure are trademarks of Johnson & Johnson Surgical Vision, Inc.  
© Johnson & Johnson Surgical Vision, Inc., 2018

**blink®**  
**intensive**  
gotas oculares calmantes/gotas oftalmológicas calmantes  
10ml

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**XXXXXX XX**

10ml INSERT (AUTO)  
207mm x 117mm  
AM01194

CE MARK SIZE/PROPORTIONS COMPLIES WITH  
EUROPEAN MEDICAL DEVICES DIRECTIVE  
93/42/EEC Annex XII.

**ENTIRE INSERT TO PRINT OUT OF PROCESS BLACK**

<b>DC • Labeling &amp; Packaging</b>																
<b>Artwork Specifications</b>																
<b>Platform:</b> MAC - Adobe Illustrator CS4	- ARTWORK IS ACTUAL SIZE. - DROP KEYLINES AND CALLOUTS BEFORE PROCESSING. - ALL COPY MUST BE CLEAR & LEGIBLE. - SMEARING/BROKEN TYPE IS NOT ACCEPTABLE.															
<b>Last Update:</b> 17/04/2018 - 12.00 v1																
<b>Note:</b> This colour copy is to show size, copy placement and colour breaks. Actual colours will be matched on press to approved colour standards and/or PMS colour swatches.																
<b>INK(S)</b>	<table border="1" style="width: 100%; border-collapse: collapse;"> <tr> <td style="width: 20%;"></td> <td style="width: 20%;"></td> <td style="width: 20%;"></td> <td style="width: 20%;"></td> <td style="width: 20%;"></td> </tr> <tr> <td></td> <td></td> <td style="background-color: black;"></td> <td></td> <td></td> </tr> <tr> <td></td> <td></td> <td style="text-align: center;">BLACK</td> <td></td> <td></td> </tr> </table>													BLACK		
		BLACK														