EN *blink*[®] intensive TRIPLE ACTION 3in1 eye drops
soothing • moisturising • protecting
blink[®] intensive TRIPLE ACTION eye drops are formulated to soothe, moisturise and protect dry-feeling or irritated eyes. Combining liposomes, sodium hyaluronate and vitamin E these TRIPLE ACTION eye drops supplement and stabilize the tear film.

Indications
 For relief from dry-feeling, sore, watery or irritated eyes. Suitable for use with contact lenses.

Instructions for use:
 Always wash and dry your hands thoroughly before using *blink*[®] intensive TRIPLE ACTION eye drops.

Shake before use
 • Instill 1 or 2 drops of *blink*[®] intensive TRIPLE ACTION eye drops in each eye
 • Blink to spread over the eye
 • Use as often as required
 • If you wear contact lenses there is no need to remove them first

Warnings:
 • In case of eye irritation, stop using the product and contact a physician
 • Do not use if you are allergic to any of the product components.
 • Do not touch the eyes with the tip of the bottle
 • Keep away from heat sources, store in a dry place between 5 and 25°C
 • Keep out of the reach of children
 • Do not use if tamper-evident seal is broken or missing
 • Do not ingest
 • Do not use the product after the expiry date
 • Use within 90 days of first opening
 • Sterile until first opening
 • Close the bottle tightly immediately after use

Contraindications: None known.
Composition: A lax coloured, sterile isotonic, buffered solution pH 7.2 containing liposomes, sodium hyaluronate, vitamin E polyethylene glycol succinate (TPGS), polyethylene glycol 400 (PEG 400), disodium edetate, polyhexanide methylguanide and purified water. Does not contain chlorhexidine, thimerosal, benzalkonium chloride, phenoxethanol.

LOT Batch code **Use by** 25°C 5°C Temperature limitation

STERILE A Sterile. Aseptic technique Do not use if package is damaged

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AM70521UN12C
 Revision Date: 07/2017
 F200 Rev. 18/07/2017

CE 0051
 Omisan farmaceutici
 Via Galilei, Guidonia M. (Rm) - Italy
 Distributed by AMO Ireland
 Block B, Liffey Valley Office Campus
 Quarryvale, Co Dublin, Ireland
blink is a trademark of Johnson & Johnson Surgical Vision, Inc.
 © Johnson & Johnson Surgical Vision, Inc. 2017

XXXXX XX

CE MARK SIZE/PROPORTIONS COMPLIES WITH EUROPEAN MEDICAL DEVICES DIRECTIVE 93/42/EEC Annex XII.

10ml INSERT
 150mm x 100mm

AM01730

ENTIRE INSERT TO PRINT OUT OF PROCESS BLACK

DC • Labeling & Packaging																	
Artwork Specifications	- ARTWORK IS ACTUAL SIZE. - DROP KEYLINES AND CALLOUTS BEFORE PROCESSING. - ALL COPY MUST BE CLEAR & LEGIBLE. - SMEARING/BROKEN TYPE IS NOT ACCEPTABLE.																
Platform: MAC - Adobe Illustrator CS4																	
Last Update: 28/07/2017 - 16.30 v2																	
Note: This colour copy is to show size, copy placement and colour breaks. Actual colours will be matched on press to approved colour standards and/or PMS colour swatches.	<table border="1"> <tr> <td>INK(S)</td> <td></td> <td></td> <td></td> <td></td> <td></td> <td></td> <td></td> </tr> <tr> <td></td> <td></td> <td></td> <td>BLACK</td> <td></td> <td></td> <td></td> <td></td> </tr> </table>	INK(S)											BLACK				
INK(S)																	
			BLACK														