blink-n-clean® 15ml

TOP FRONT

(EN) For clearer, more comfortable contact lenses during wear

blink-n-clean® drops are formulated to give contact lens wearers clearer, more comfortable lenses throughout the day without the need to remove them first. This gentle solution does more than just re-wet; it washes away natural debris and protein build up to leave lenses clear, fresh and comfortable. Particularly useful for extended wear lenses or when wearing lenses for longer periods of time.

NO NEED TO REMOVE LENSES

Directions for use:

- Apply one or two drops to each eye up to four times daily
- Blink to wash the lens

Precautions:

All instructions must be followed. Failure to do so may result in serious eye infection or injury.

- Do not use if tamper-evident seal is broken or missing
- Close the bottle tightly immediately after use
- Do not use if allergic to any of the ingredients
- Do not use more than four times a day
- Do not drink the solution
- Store between 5°C and 25°C
- Keep out of reach of children
- Do not use after the expiry date
- Discard any remaining solution 60 days after first opening the bottle
- To avoid contamination do not touch bottle tip on any surface

Contains:

Sodium Chloride
Tromethamine
Hydroxypropylmethylcellulose
Tyloxapol
EDTA
Polyhexamethylene biguanide 0.0001%
Purified water

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CE 0123 **STERILE A**

AMO Ireland
Block B, Liffey Valley Office Campus
Quarryvale, Co. Dublin, Ireland
blink-n-clean is a trademark of Johnson & Johnson Surgical Vision, Inc.
© Johnson & Johnson Surgical Vision, Inc., 2018

CE mark size/proportions complies with European Medical Devices Directive 93/42/EEC Annex XII

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XXXXX XX

10ml INSERT (AUTO)
207mm x 117mm
AM01194

ENTIRE INSERT TO PRINT OUT OF PROCESS BLACK

DC • Labeling & Packaging																					
Artwork Specifications																					
Platform: MAC - Adobe Illustrator CS4	- ARTWORK IS ACTUAL SIZE. - DROP KEYLINES AND CALLOUTS BEFORE PROCESSING. - ALL COPY MUST BE CLEAR & LEGIBLE. - SMEARING/BROKEN TYPE IS NOT ACCEPTABLE.																				
Last Update: 24/04/2018 - 14.00 v1																					
Note: This colour copy is to show size, copy placement and colour breaks. Actual colours will be matched on press to approved colour standards and/or PMS colour swatches.	<table border="1"> <tr> <td>INK(S)</td> <td></td> <td></td> <td></td> <td></td> <td></td> <td></td> <td></td> <td></td> <td></td> </tr> <tr> <td></td> <td></td> <td></td> <td></td> <td>BLACK</td> <td></td> <td></td> <td></td> <td></td> <td></td> </tr> </table>	INK(S)														BLACK					
INK(S)																					
				BLACK																	