**(EN) blink® refreshing hydrating eye mist 10ml**  
**For relief from dry-feeling, tired or irritated eyes**  
 blink® refreshing hydrating eye mist is a biocompatible lubricating, moisturizing and refreshing ophthalmic solution.  
**Indications:**  
 Spray onto the open eye for relief from dry-feeling, tired or irritated eyes caused by prolonged computer use, driving, contact lens wear, insufficient tearing, smoky or air conditioned environments, exposure to wind and dust. Suitable for use with contact lenses.  
**Instructions for use:**  
 • Hold the bottle firmly, without covering the spray opening, about 10cm directly to the side of your eye  
 • Look straight ahead, press firmly twice per eye  
 • Use as often as required  
 • If you wear contact lenses there is no need to remove them first  
**Symbols**  
 Consult instructions for use Manufacturer Batch code Use by 25°C Temperature limitation  
 Sterile, Aseptic technique Do not use if package is damaged

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CE 0051 AM70408UN11C  
 Revision Date: 07/2017  
 F168 Rev. 18/07/2017  
 Omisan pharmaceutical  
 Via Galilei, Guidonia M. (RM) - Italy  
 Distributed by AMO Ireland  
 Block B, Liffey Valley Office Campus  
 Quarryvale, Co Dublin, Ireland  
 blink is a trademark of Johnson & Johnson Surgical Vision, Inc.  
 © Johnson & Johnson Surgical Vision, Inc. 2017  
 XXXXX XX

CE MARK SIZE/PROPORTIONS COMPLIES WITH EUROPEAN MEDICAL DEVICES DIRECTIVE 93/42/EEC Annex XII.

10ml INSERT  
 150mm x 100mm

AM01730

ENTIRE INSERT TO PRINT OUT OF PROCESS BLACK

<b>DC • Labeling &amp; Packaging</b>																					
<b>Artwork Specifications</b>																					
<b>Platform:</b> MAC - Adobe Illustrator CS4	- ARTWORK IS ACTUAL SIZE. - DROP KEYLINES AND CALLOUTS BEFORE PROCESSING. - ALL COPY MUST BE CLEAR & LEGIBLE. - SMearing/BROKEN TYPE IS NOT ACCEPTABLE.																				
<b>Last Update:</b> 17/08/2017 - 14.00 v3																					
<b>Note:</b> This colour copy is to show size, copy placement and colour breaks. Actual colours will be matched on press to approved colour standards and/or PMS colour swatches.	<table border="1"> <tr> <td>INK(S)</td> <td></td> <td></td> <td></td> <td></td> <td></td> <td></td> <td></td> <td></td> <td></td> </tr> <tr> <td></td> <td></td> <td></td> <td></td> <td>BLACK</td> <td></td> <td></td> <td></td> <td></td> <td></td> </tr> </table>	INK(S)														BLACK					
INK(S)																					
				BLACK																	