

**PLEASE READ CAREFULLY AND KEEP THIS PACKAGE INSERT
FOR FUTURE USE IN CASE YOU HAVE A PROBLEM.**

BAUSCH & LOMB Sensitive Eyes® Daily Cleaner

CONTENTS: Bausch & Lomb® Sensitive Eyes Daily Cleaner is a sterile, isotonic, buffered solution that contains hydroxypropyl methylcellulose, poloxamine, sodium borate and sodium chloride; preserved with edetate disodium (0.5%) and sorbic acid (0.25%).

ACTIONS: Loosens and removes accumulations of film, deposits and debris from your lenses.

INDICATIONS (Uses): Bausch & Lomb Sensitive Eyes Daily Cleaner is indicated for use in cleaning soft contact lenses, in conjunction with thermal (heat), chemical (not heat) and hydrogen peroxide disinfection regimens. This cleaner may be used with extended wear soft contact lenses as often as daily or as recommended by your eye care practitioner.

CONTRAINDICATIONS (Reasons not to use): If you are allergic to any ingredient in Bausch & Lomb Sensitive Eyes Daily Cleaner, do not use this product.

WARNINGS: PROBLEMS WITH CONTACT LENSES AND LENS CARE PRODUCTS COULD RESULT IN SERIOUS INJURY TO THE EYE. It is essential that you follow your eye care practitioner's directions and all labeling instructions for proper use and care of your lenses and lens care products, including the lens case.

Eye problems, including corneal ulcers, can develop rapidly and lead to loss of vision.

Daily wear lenses are not indicated for overnight wear and should not be worn while sleeping. Clinical studies have shown the risk of serious adverse reactions is increased when the lenses are worn overnight.

Extended wear lenses should be regularly removed for cleaning and disinfection or for disposal and replacement on the schedule prescribed by your eye care practitioner. Clinical studies have shown that there is an increased incidence of serious adverse reactions in extended wear contact lens users as compared to daily wear contact lens users. Studies have also shown that the risk of serious adverse reactions increases the longer extended wear lenses are worn before removal for cleaning and disinfection or for disposal and replacement.

Studies have also shown that smokers have a higher incidence of adverse reactions.

If you experience eye discomfort, excessive tearing, vision changes, redness of the eye, immediately remove your lenses and promptly contact your eye care practitioner.

It is recommended that contact lens wearers see their eye care practitioner twice each year or, if directed, more frequently.

- To avoid contamination, do not touch tip of container to any surface. Replace cap after using.
- Not intended for use directly in the eye.

PRECAUTIONS:

- Keep the bottle tightly closed when not in use.
- Store at room temperature.
- Use before the expiration date marked on the bottle and carton.

ADVERSE REACTIONS (Problems and what to do): The following problems may occur: eyes sting, burn or itch (irritation), comfort is less than when lens was first placed on the eye, feeling of something in the eye (foreign body, scratched area), excessive watering (tearing) of the eye, unusual eye secretions, redness of the eye, reduced sharpness of vision (poor visual acuity), blurred vision, rainbows or halos around objects, sensitivity to light (photophobia), or dry eyes.

If you notice any of the above:

- **IMMEDIATELY REMOVE YOUR LENSES.**
- If the discomfort or problem stops, then look closely at the lens.
- If the lens is in any way damaged, DO NOT put the lens back on your eye. Place the lens in the storage case and contact your eye care practitioner.
- If the lens has dirt, an eyelash, or other foreign body on it, or the problem stops and the lens appears undamaged, thoroughly clean, rinse, and disinfect the lens, then reinsert it.
- If the problem continues, **IMMEDIATELY** remove the lens and consult your eye care practitioner.

If any of the above symptoms occur, a serious condition such as infection, corneal ulcer, neovascularization or iritis may be present. Seek immediate professional identification of the problem and prompt treatment to avoid serious eye damage.

GOOD LENS CARE PRACTICES:

- Always wash and rinse your hands before you handle your lenses.
- Clean, rinse and disinfect your lenses each time you remove them.
- Always handle the same lens, the right or left, first in order to avoid mix-ups.
- After use, always empty and rinse lens case with fresh rinsing solution and allow to air dry.

DIRECTIONS:

- After removing the lens, place it in the palm of your hand, and place three drops of Sensitive Eyes Daily Cleaner on each lens surface.
- Rub the lens for 20 seconds either in the palm of your hand using the forefinger of the other hand, or between the thumb and forefinger. Be sure to clean each side of the lens.
- Rinse the lens thoroughly by holding the lens in a steady stream of Bausch & Lomb Sensitive Eyes Saline Solution for approximately 10 seconds.
- Disinfect using the procedure which your practitioner has recommended.

NOTE: This product is not intended to remove protein deposits. Use an enzymatic cleaning tablet, such as Bausch & Lomb Sensitive Eyes Enzymatic Cleaner, as directed by your eye care practitioner.

HOW SUPPLIED: Bausch & Lomb Sensitive Eyes Daily Cleaner is available for purchase in sterile .66 Fl. Oz. (20ml) plastic bottles. Bottles and cartons are marked with a lot number and expiration date.

PRODUCT SATISFACTION GUARANTEE
If you are not completely satisfied with this product, call the toll-free number on this insert.

Toll Free Product Information

Call: 1-800-553-5340

Bausch & Lomb and Sensitive Eyes are registered trademarks of Bausch & Lomb Incorporated.

© Bausch & Lomb Incorporated

Manufactured by:

Bausch & Lomb

Rochester, NY 14692-0450

Made in USA